CZ Jak používat

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EN How to use

- 1) Make sure your Pinktens is turned off before applying or removing electrodes.
- 2) Electrodes do not stick well if any lotion, cream, oil, etc is left on the skin. Clean and dry the skin prior to each application.
- 3) Clip the cable with the electrodes prior to application.
- 4) Place the electrodes following the positioning instructions proposed on the App. Press firmly from the centre of the electrodes to the outer edges, the full surface of the electrodes must be in contact with your skin. Adhesion improves when electrodes reach skin temperature.
- 5) Place the electrodes back on the right side of the protective plastic liner after use and put them back in the sealed bag to prevent gel drying out. Do not take out the electrodes by pulling on the cable, lift a corner of the electrode and slowly peel it off the skin, touching the adhesive gel as little as possible.

- 6) The electrodes are intended for single patient use only. The number of application depends on your skin condition, anatomical placement, type of treatment and care after use. If adhesive gel becomes dry you can add one or two drops of ordinary tap water spread evenly and gently with your fingertips. Do not use too much water as this may reduce the life of the gel.
- 7) Store the electrodes bag at room or cool temperature (+5 - +25 °C) and keep out from direct sunlight.

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