
Sinus Inhaler

with VapoPads

VH200E





GB	SINUS INHALER	4
	User instructions	
DE	SINUS INHALATOR	8
	Ultraschall-Kaltluft	
DK	BIHULEINHALATOR	14
	Brugervejledning	
ES	INHALADOR NASAL	18
	Instrucciones de uso	
FI	HÖYRYHENGITYSLAITE	21
	Käyttöohjeet	
FR	INHALATEUR POUR SINUS	25
	Mode d'emploi	
HR	INHALATOR ZA SINUSE	30
	Korisničke upute	
HU	INHALÁTOR	34
	Használati utasítás	
IT	INALATORE	38
	Istruzioni per l'uso	
NL	INHALATOR	42
	Gebruikersinstructies	
NO	BIHULE-INHALATOR	47
	Bruksanvisning	
PL	INHALATOR ZATOKOWY	51
	Instrukcja obsługi	
PT	INALADOR DOS SEIOS NAsAIS	56
	Instruções do utilizador	
SA	منشقة الجيوب الأنفية	63
	تعليمات المستخدم	
SE	SINUS INHALATOR	64
	Bruksanvisning	
TR	SİNÜS NEBULİZATÖR	68
	Kullanım Talimatları	
CS	SINUS INHALER - Sinusový inhalátor	72
	Návod k obsluze	
ZA	ADDITIONAL IMPORTANT SAFETY INSTRUCTIONS	76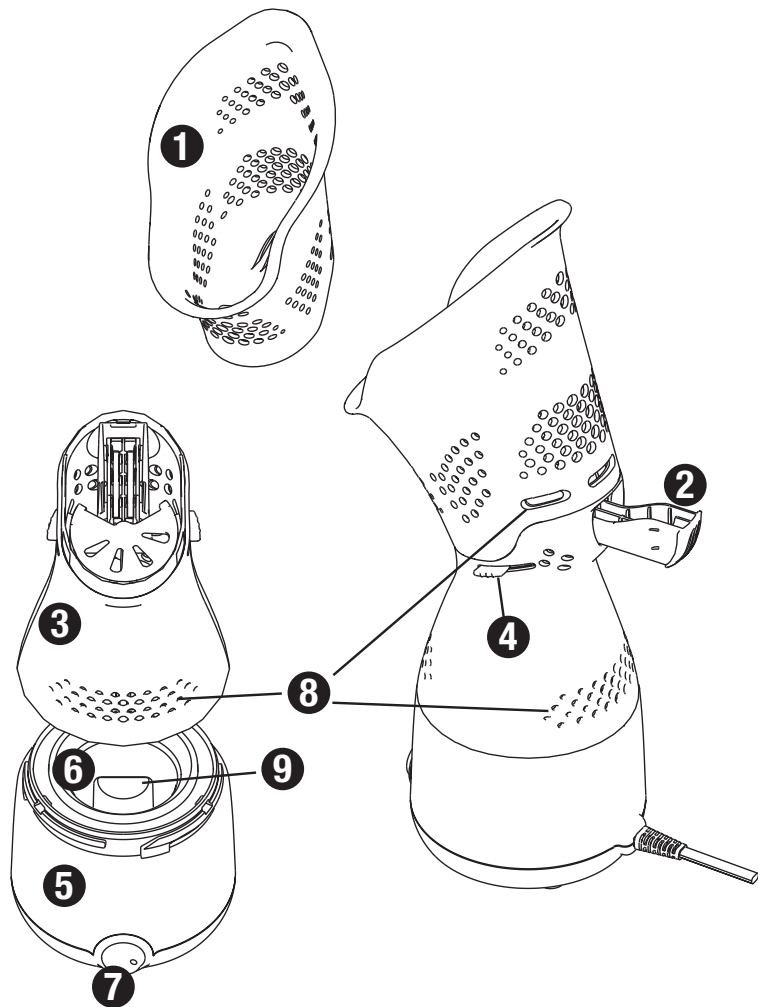




COMPONENTS

- | | | |
|------------------------|------------------|--------------------|
| 1. Mask | 2. VapoPads Tray | 3. Top Housing |
| 4. Steam Disk Controls | 5. Base | 6. Water Reservoir |
| 7. Power Switch | 8. Vents | 9. Heating Element |




A)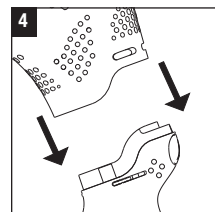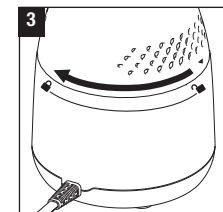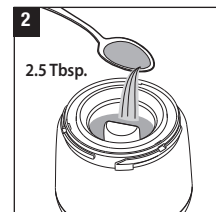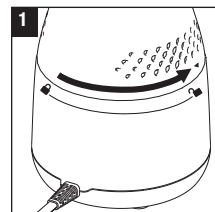

B)

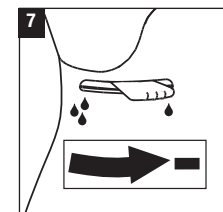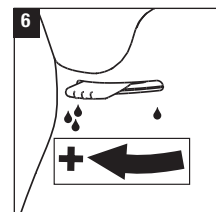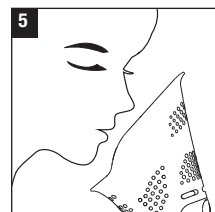C)

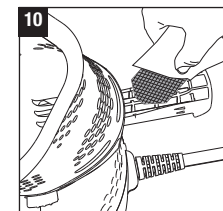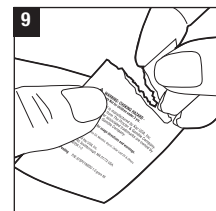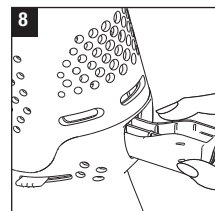Use only with VapoPads
Rosemary&Lavender (VBR7)
or Menthol (VH7)

IMPORTANT SAFETY INSTRUCTIONS

-  Read through all the instructions before starting to use the appliance. Keep this manual in a safe place for future reference.

These instructions are also available on our website. Please visit www.hot-europe.com.

WARNING:

- Read all instructions before using this inhaler.
- The appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of this appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Unplug the appliance during filling and cleaning
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Always place this inhaler on a firm, flat and level surface. This inhaler may not work properly on an uneven surface.
- Place this inhaler in an area where it is not accessible to children.
- Do not place the inhaler near

any heat sources such as stoves, radiators, and heaters.

- Do not use the inhaler outdoors.
- To disconnect the inhaler, first turn the device OFF, then grip the plug and pull it from the wall outlet. Never pull by the cord.
- Never tilt or attempt to move the inhaler while it is operating or filled with water. UNPLUG the inhaler before moving.
- This inhaler requires regular cleaning. Refer to the CLEANING instructions provided. Never clean the inhaler in any manner other than as instructed in this manual.
- Do not place the product directly on wood furniture or other surfaces that could be damaged by water or liquids. The manufacturer will not accept responsibility for property damage caused by spillage or leaking.

COMPONENTS (See page 2)

1. Mask
2. VapoPads Tray
3. Top Housing
4. Steam Disk Controls
5. Base
6. Water Reservoir
7. Power Switch
8. Vents
9. Heating Element

FIRST TIME OF USE

Before using the unit for the first time, you should note the following points:

- Remove all packing material from the unit
- It is recommended that you disinfect your Inhaler prior to first use. See cleaning instructions.
- Select a firm, level location at least 15 cm from any walls for proper airflow.
- Place the Inhaler on a water resistant surface.

OPERATING INSTRUCTIONS

FILLING (See appendix A on page 3)

- Fully unravel cord. Failure to do so could cause overheating and possible fire hazard. DO NOT use with extension cords or power strip. Place Inhaler on a flat, stable surface for use.
- Unlock the Top Housing from the Base by twisting the Top Housing counter clockwise (1). When fully unlocked, the arrow  on the Top Housing will align with the unlock  symbol on the Base. Lift the Top Housing straight up and set aside.
- Fill Water Reservoir with up to 2.5 tablespoons (40 ml) of cool tap water (2). DO NOT fill above Heating Element. Use with water only. DO NOT add any essential oils, medication or inhalant liquids to the Water Reservoir. An unsafe condition may result if anything is added to the water.
- Place Top Housing on the Base and twist clockwise to lock (3). When fully locked, you will feel a click and the arrow  on the Top Housing will align with the lock  symbol on Base.
- Place Mask on Top Housing and gently push down to ensure Mask is secure (4).

CAUTION: NEVER move, lift, shake or tilt the Inhaler while it is in use or while it contains hot water. DO NOT remove the Mask or Top Housing after turning ON the Inhaler. Steam is produced by boiling water; tilting or shaking unit can cause water to spill and possible burns. DO NOT block any vents.

CAUTION: Do not fill above 

POWER

(see appendix B on page 3)

CAUTION: DO NOT PLUG IN THE INHALER WITH WET HANDS AS AN ELECTRICAL SHOCK COULD OCCUR.

CAUTION: Hot steam  

- Connect cord to electrical outlet. DO NOT use with extension cord or power strip. Press the On/Off Switch to the ON position. The indicator light will illuminate.
- The Steam Inhaler will take approximately 5 minutes to begin making steam. Use caution, the Water Reservoir and water will become hot.
- Place your face over the Mask (5). DO NOT rest your face on the Mask. Adjust the distance to your temperature skin sensitivity. Breathe normally through your nose and mouth, inhaling steam into your nasal and throat passages. Continue treatment for 5 to 20 minutes.

ADJUSTING STEAM INTENSITY

- **For More Steam:** Move the Steam Disk Control clockwise for maximum steam (6).
- **For Less Steam:** Move the Steam Disk Control counterclockwise for minimum steam (7). If at any time the treatment becomes too intense, stop treatment immediately.
- When treatment is complete, move Power Switch to the OFF position. The indicator light will go out. Disconnect plug from electrical outlet. Allow the appliance to cool for at least 20 minutes before moving or refilling it (see Care and Cleaning section). DO NOT operate Steam Inhaler when empty.

NOTE: Steam Inhaler will automatically shut-off when water runs out. If this happens, move Power Switch to OFF position and unplug unit.

SCENT PAD DIRECTIONS FOR USE (see appendix C on page 3)

Your Steam Inhaler can be used with VapoPads, to provide up to 8 hours of soothing vapors. Two free samples of the menthol VapoPads are included with your device.

To use your unit with scent pads, ensure that the unit is OFF and follow these instructions:

- Gently slide out the VapoPads tray from the back of the Top Housing (8). Note: Only slide tray out. DO NOT attempt to detach the tray from the Top Housing.
- Open VapoPad sachet by tearing notch on pad bag (9). DO NOT touch pad with hands. If pad is touched, do not rub face or eyes as it may cause irritation.
- Insert VapoPad with angled end at the front of the tray (2). Up to two scent pads can be inserted, one in each slot of the VapoPad tray. Push VapoPads Tray in to close.
- When the VapoPad is dried and no longer providing scent, discard.

NOTE: DO NOT operate Steam Inhaler if VapoPads Tray is missing or not secured.

This Steam Inhaler (with or without VapoPads) is not recommended for use by children.

WARNING:

Pad May present a choking hazard. Keep out of reach of children and pets.

H317 May cause an allergic skin reaction. P102 Keep out of reach of children. P302 + P352 IF ON SKIN: wash with plenty of soap and water. P333 + P313 If skin irritation or rash occurs: Get medical advice / attention
Ingredients: eucalyptus oil, menthol, glycol, cedar leaf oil & others

2 menthol VapoPads included.

WARNING: Choking hazard, keep out of reach of children and pets. Do not use for babies less than 36 month old, for more than 16 hours in a 24 hour period and/or not more than 2 pads in a 24 hour period.

CARE AND CLEANING INSTRUCTIONS

CAUTION: ALWAYS disconnect the plug from the electrical outlet and let the Steam Inhaler cool down for at least 20 minutes before cleaning.

AFTER EACH USE

- Remove Mask and Top Housing and wipe dry with a clean cloth or paper towel.
- If any water is left in the Water Reservoir, pour water out and wipe Water Reservoir and Base with a dry, clean cloth or paper towel. NEVER immerse the Base in water or any other liquid.
- Replace Top Housing and Mask; store in a cool, dry place.

WEEKLY CLEANING

- Remove Mask and Top Housing.
- Pour 2.5 tablespoons (40ml) of undiluted distilled white vinegar into the Water Reservoir and let sit for 10 minutes.
- Empty the distilled white vinegar and rinse out Water Reservoir with fresh, clean tap water. Use caution to not let water get into the electrical areas.
- Dry the Water Reservoir and Base with a clean cloth or paper towel.
- Wash Mask with mild soap and warm water then carefully rinse off all soap with fresh, clean tap water. Dry with a clean cloth or paper towel.
- Replace Top Housing and Mask. Store in a cool, dry place.
- To clean the VapoPads Tray, wet the tip of a cotton swab and wipe slots.

NOTE: The Top Housing is dishwasher safe. It may be cleaned in the top rack of the dishwasher or by hand as noted in the instructions. DO NOT place in the bottom rack of dishwasher. DO NOT attempt to disassemble the Top Housing. The Heating Element should NEVER be put in the dishwasher.

STORAGE

Follow cleaning instruction and dry product thoroughly.

DISPOSAL

This appliance complies with EU legislation 2012/19/EU on end of life recycling. Products showing the 'Crossed Through' wheeled bin symbol on either the rating label, gift box or instructions must be recycled separately from household waste at the end of their useful life.

Please DO NOT dispose of appliance in normal household waste. Your local appliance retailer may operate a 'take-back' scheme when you are ready to purchase a replacement product, alternatively contact your local government authority for further help and advice on where to take your appliance for recycling.

GUARANTEE

Please read all instructions before attempting to use this device. Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant guarantee period. Any claim under the guarantee will not be valid without a proof of purchase.

Your device is guaranteed for two Years (2 Years) from date of purchase.

This guarantee covers defects in materials or workmanship that occur under normal use; defective devices meeting these criteria will be replaced free of charge.

The guarantee DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions. The guarantee becomes void if the device is opened, tampered with or used with other parts or accessories, or if repairs are undertaken by unauthorised persons.

Accessories and consumables are excluded from any warranty.

For support requests, please visit www.hot-europe.com/support or find service contact information at the end of this owner's manual.

This Guarantee is applicable to Europe, Russia, Middle East and Africa only.

UK Only: This does not affect your consumer statutory rights.

The LOT of your device are printed on the rating label on the packaging of the product.

CONDITION OF SALE

As a condition of sale, the purchaser assumes responsibility for the proper use and care of this appliance in accordance with these printed instructions. The purchaser or user must judge himself or herself when to use the appliance and the length or use.

NOTE: IF YOU EXPERIENCE A PROBLEM WITH YOUR INHALER PLEASE SEE WARRANTY FOR INSTRUCTIONS. PLEASE DO NOT ATTEMPT TO REPAIR THE INHALER YOURSELF. DOING SO WILL VOID THE WARRANTY AND COULD CAUSE DAMAGE OR PERSONAL INJURY.

This product carries the CE mark and is manufactured in conformity with the Electromagnetic Directive 2014/30/EU, the Low Voltage Directive 2014/35/EU and the RoHS Directive 2011/65/EU.

Technical specifications are subject to change.

VH200E

220-240V ~ 50-60Hz
70W

TROUBLE SHOOTING

PROBLEM	POSSIBLE CAUSE	CORRECTIVE ACTION
Power switch is set to "on" and Power Light is off	Unit not plugged in.	Plug unit in.
	No power at outlet.	Check circuits, fuses, try a different outlet.
Water overflows from Water Reservoir	Unit has been filled above max fill line.	Empty unit. dry and then refill taking care not to fill past the max fill line.
Inhaler produces only little mist or no mist	No water in Water Reservoir.	Fill the Water Reservoir.
	The unit is not placed on a firm, level surface.	Place the Inhaler on a firm, level surface.
	Mineral deposits on Heating Element.	Clean or disinfect the Heating Element.
	The Water Reservoir was washed with a washing liquid and not rinsed thoroughly.	Empty the water and rinse thoroughly with warm water.

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[ZA] ADDITIONAL IMPORTANT SAFETY INSTRUCTIONS

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.

Support contact numbers

Austria

☎ +43 (0)1 360 277 1225

Bahrain

☎ +973 1725 1988

Belgium/Luxemburg

☎ +32 (0) 2 620 01 01

Bosnia

☎ +387 (33) 636 285

Bulgaria

☎ +359 (2) 40 24 600

Croatia

☎ +385 (01) 3444 856

Cyprus

☎ +357 22575016

Cyprus Northern region

☎ +0392 22 72 367

Czech Republic

☎ +420 (0) 2 25 43 97 69

Denmark

☎ +45 35 15 80 40

Finland

☎ +358 (0)9 81 71 00 14

France

☎ +33 (0) 1 85 14 80 95

Germany

☎ +49 (0) 21 173 749 003

Greece

☎ +30 211 180 94 56

Hungary

☎ +36 (06) 1 429 2216

Iceland

☎ +35 852 07 900

Israel

☎ +1 800250221

Italy

☎ +39 02 3859 1183

Jordan

☎ +962 6 582 0112-3-4-5

Kuwait

☎ +965 1 88 55 22

Lebanon

☎ +961 1 696 999

Netherlands

☎ +31 (0) 78 201 8001

Norway

☎ +47 23 50 01 20

Poland

☎ +48 22 512 39 02

Portugal

☎ +351 21 060 8045

Qatar

☎ +974 4437 3644

Romania

☎ +40 (0) 264 406488

Saudi-Arabia

☎ +966 3 8692244

Western Region - Jeddah

☎ +966 (0) 22565555

Central Region - Riyadh

☎ +966 (0) 12886808

Eastern Region - Al-Khobar

☎ +966 (0) 38940555

South Region - Abha

☎ +966 (0) 72376062

Slovakia

☎ +421 2 5011 2115

Slovenia

☎ +386 (01) 588 68 00

South Africa

☎ +27 11 844 6190

Spain

☎ +34 913 754 176

Sweden

☎ +46 (0)8 5199 3097

Switzerland

☎ +41 (0) 22 567 5200

Tunisia

☎ +216 71716880

Turkey

☎ +90 216 337 22 55

UAE

☎ +971 4 269 1230

United Kingdom/Ireland

☎ +44 115 96 57 430

Certain trademarks used under license from The Procter & Gamble Company or its affiliates.

© 2019 All rights reserved.

Kaz Europe Sàrl
A Helen of Troy Company
Place Chauderon 18
CH-1003 Lausanne - Switzerland
www.hot-europe.com

Made and printed
in China

P/N: A003177R0
07JAN19

Helen of Troy

CREATIVE DEPARTMENT ARTWORK SPECIFICATIONS

Brand: Vicks
Category: Inhaler
Model: **VH200E2-E5**
Artwork Part #: A003177R0
Die Line Part #: N/A
Subject: Owner's Manual
Region: EMEA
Flat Size: MM: W240x H200
Folded Size: MM: W120 x H200
Scale: 1/1
Material: 50# gloss or matte text
Page Count: 76
Revision: 2
Date: **07JAN19**
Release Date: 21JAN19
Re-release Date:
Re-release Date:

Colors:

Dielines (Do not print)

Spot Colors

Cyan
0%

Magenta
0%

Yellow
0%

Black
100%

PMS
XXXX

PMS
XXXX

Special Instructions:

**Quality Requirement of Artwork and Quality
Clarification Process of Artwork Printing:
Meet Eng-QS-06&02**

Helen of Troy
Creative Services
Marlborough, MA 01752, USA
+1 508 490 7000